

SpandaGrip™

Tubular Elastic Support Bandage

Latex-Free

Provides Sustained Pressure & Support for: Strains & Sprains, Soft Tissue Injuries, Support following Cast Removal, General Edema, Post-Graft Pressure for Burn Patients & Rib Cage Injuries

INSTRUCTIONS FOR USE



1. CUT A LENGTH OF SpandaGrip™ TO FIT THE LIMB OR AREA TO BE COVERED - ALLOW AN EXTRA 2-3cm (1 inch) FOR OVERLAP
2. DRAW THE BANDAGE CAREFULLY OVER THE LIMB OR AREA LIKE A STOCKING
3. A SINGLE LAYER ONLY IS RECOMMENDED

SPANDAGRIP TUBULAR ELASTIC BANDAGE - COMPRESSION SIZING CHART

SINGLE LAYER APPLICATION

NATURAL			Low Pressure 5 - 10 mmHg		Medium Pressure 10 - 20 mmHg		High Pressure 20 - 30 mmHg	
PRODUCT NUMBER	Color Code	SIZE	Low Pressure Circumference Centimeters	Circumference Inches	Medium Pressure Circumference Centimeters	Circumference Inches	High Pressure Circumference Centimeters	Inches
SAG13110	Purple	1.50" (A)	19 - 22 cm	7.5 - 8.7 in	22 - 24 cm	8.7 - 9.4 in	24 - 27 cm	9.4 - 10.6 in
SAG13111	Olive Green	2.50" (B)	22 - 24 cm	8.7 - 9.4 in	24 - 27 cm	9.4 - 10.6 in	27 - 30 cm	10.6 - 11.8 in
SAG13112	Yellow	2.75" (C)	24 - 27 cm	9.4 - 10.6 in	27 - 30 cm	10.6 - 11.8 in	30 - 35 cm	11.8 - 13.8 in
SAG13113	Light Blue	3.00" (D)	27 - 30 cm	10.6 - 11.8 in	30 - 35 cm	11.8 - 13.8 in	35 - 44 cm	13.8 - 17.3 in
SAG13114	Brown	3.50" (E)	30 - 35 cm	11.8 - 13.8 in	35 - 44 cm	13.8 - 17.3 in	44 - 48 cm	17.3 - 18.9 in
SAG13115	Aqua	4.00" (F)	35 - 44 cm	13.8 - 17.3 in	44 - 58 cm	17.3 - 22.8 in	58 - 64 cm	22.8 - 25.2 in
SAG13116	Pink	4.50" (G)	44 - 58 cm	17.3 - 22.8 in	58 - 64 cm	22.8 - 25.2 in	64 - 72 cm	25.2 - 28.3 in
SAG13121	Dark Blue	5.50" (I)	58 - 64 cm	22.8 - 25.2 in	64 - 72 cm	25.2 - 28.3 in	72 - 86 cm	28.3 - 33.8 in
SAG13117	Orange	6.75" (J)	64 - 72 cm	25.2 - 28.3 in	72 - 86 cm	28.3 - 33.8 in	86 - 96 cm	33.8 - 37.8 in
SAG13118	Red	8.25" (K)	72 - 86 cm	28.3 - 33.8 in	86 - 96 cm	33.8 - 37.8 in	96 - 108 cm	37.8 - 42.5 in

PRODUCT NUMBER	Color Code	SIZE	Low Pressure Circumference		Medium Pressure Circumference		High Pressure Circumference	
			Centimeters	Inches	Centimeters	Inches	Centimeters	Inches
SAG13120	Black	5.00" (H)	53 - 62 cm	20.9 - 24.4 in	62 - 70 cm	24.4 - 27.6 in	70 - 83 cm	27.6 - 32.7 in
SAG13119	Light Green	13.00" (L)	105 - 130 cm	41.3 - 51.2 in	130 - 149 cm	51.2 - 58.7 in	149 - 160 cm	58.7 - 63 in

SpandaGrip™ has been designed for tissue support and provides sustained pressure. It does not provide graduated therapeutic compression. The application of SpandaGrip is solely at the discretion of the treating physician or other certified clinical personnel and should not be utilized without considering the clinical presentation of the patient and the facility's protocols and guidelines. Any information, sizing charts or other sizing instruments provided act merely as guidelines to assist in the proper application of SpandaGrip, not as recommended usage for any patient or clinical condition.

