

SPANDAGRIP™

TUBULAR ELASTIC SUPPORT BANDAGE



USAGE GUIDELINES

The compression and support properties of SpandaGrip remain consistent even after reapplication.

- Generally SpandaGrip can be reapplied 20 times or more (conservative estimate) depending upon the type of condition it is being used for. Support for strains, sprains, soft tissue injury and general edema being the most general.
- If used for wound dressing securement it might get soiled which will affect how many times it can be reapplied.
- If SpandaGrip is badly soiled it should be laundered frequently or be replaced.
- SpandaGrip can be sterilized by autoclaving. Recommended Autoclave Cycle: 275° Fahrenheit for 4 minutes.
- The RH/Relative Humidity of SpandaGrip is 60% - 70%. There is no anticipated humidity reduction that will adversely affect SpandaGrip.

Medi-Tech International Corp.
(800) 333-0109
www.medi-techintl.com